Contents

Owner/User Responsibility 3 Physician Responsibility 3 Replacement Z1 Parts and Supplies 3 Terms Used in this User Guide 4 Medical Information 4 Z1 Warnings and Cautions 5 Battery Warnings and Cautions 8 Getting to Know Your Z1 System 10 Package Contents - Z1 CPAP Base Model 10 Package Contents - PowerShell Module 10 Z1 Unplugged 10 Z1 CPAP Device 11 Tube Adapter 11 CPAP Tube 11 Filter 11 USB Micro Cable 11 USB Micro Cable 11 USB Micro Cable 11 Notsture Exchanger (HME) 11 PowerShell (Optional Accessory) 11 Initial Set Up 12 Unpacking the Z1 Box. 12 Joing Your Z1 15 Starting and Stopping Treatment. 17 Programming Treatment Pressure 18 User Software 19 USB Firmware Upgrade. 19 USB Firm	Welcome to Your Z1 CPAP System	3
Replacement Z1 Parts and Supplies3Terms Used in this User Guide4Medical Information4Z1 Warnings and Cautions5Battery Warnings and Cautions8Getting to Know Your Z1 System10Package Contents - Z1 CPAP Base Model10Package Contents - PowerShell Module10Z1 Unplugged10Z1 CPAP Device11Tube Adapter11End Cap.11Filter11UsB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Upacking the Z1 Box.12Assembling and Powering Up13Using Your Z115Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode19Ubs Firmware Upgrade.19User Software21Unspections21Care21Inspections21Cleaning22Troubleshooting23Guidance and Manufacturer's Declaration23Electromagnetic Emissions & Immunity25	Owner/User Responsibility	3
Terms Used in this User Guide4Medical Information4Z1 Warnings and Cautions5Battery Warnings and Cautions8Getting to Know Your Z1 System10Package Contents - Z1 CPAP Base Model10Package Contents - PowerShell Module10Z1 Unplugged10Z1 CPAP Device11Tube Adapter11End Cap.11Filter11End Cap.11Filter11Filter11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Uppacking the Z1 Box12Assembling and Powering Up13Using Your Z115Starting and Stopping Treatment17Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections22Troubleshooting22Troubleshooting23Guidance and Manufacturer's Declaration23Electromagnetic Emissions & Immunity25	Physician Responsibility	3
Medical Information 4 Z1 Warnings and Cautions 5 Battery Warnings and Cautions 8 Getting to Know Your Z1 System 10 Package Contents - Z1 CPAP Base Model 10 Package Contents - PowerShell Module 10 Z1 Unplugged 10 Z1 CPAP Device 11 Tube Adapter 11 End Cap. 11 Filter 11 End Cap. 11 Filter 11 PowerShell (Optional Accessory) 11 Initial Set Up 12 Unpacking the Z1 Box 12 Assembling and Powering Up 13 Using Your Z1 15 LCD Display 15 Starting and Stopping Treatment 17 Programming Ramp Mode 18 User Software 19 Charging the Battery (PowerShell Only) 20 Care 21 Inspections 21 Inspections 21 Inspections 21 Inspections 22 Guidance and Manufacturer's	Replacement Z1 Parts and Supplies	3
Z1 Warnings and Cautions5Battery Warnings and Cautions8Getting to Know Your Z1 System10Package Contents - Z1 CPAP Base Model10Package Contents - PowerShell Module10Z1 Unplugged10Z1 Unplugged10Z1 Unplugged11Tube Adapter11CPAP Tube11End Cap.11Filter11New Shell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box12Unpacking the Z1 Box12Unpacking Your Z115LCD Display15Starting and Stopping Treatment.17Programming Ramp Mode19USB Firmware Upgrade.19User Software19USB Firmware Upgrade.21Inspections21CPAP Tube Replacement22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.23Guidance and Manufacturer's Declaration24Electromagnetic Emissions & Immunity25	Terms Used in this User Guide	4
Battery Warnings and Cautions8Getting to Know Your Z1 System10Package Contents - Z1 CPAP Base Model10Package Contents - PowerShell Module10Z1 Unplugged10Z1 Unplugged11Tube Adapter11CPAP Device11End Cap.11Filter11USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Unpacking Your Z115LCD Display15Starting and Stopping Treatment.17Programming Ramp Mode18User Software19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.23Guidance and Manufacturer's Declaration23Electromagnetic Emissions & Immunity25	Medical Information	4
Getting to Know Your Z1 System10Package Contents - Z1 CPAP Base Model10Package Contents - PowerShell Module10Z1 Unplugged10Z1 CPAP Device11Tube Adapter11CPAP Tube11End Cap.11Filter11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box12Unpacking and Powering Up13Using Your Z115Starting and Stopping Treatment17Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21Cleaning22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.23Guidance and Manufacturer's Declaration24Electromagnetic Emissions & Immunity25	Z1 Warnings and Cautions	5
Package Contents - Z1 CPAP Base Model10Package Contents - PowerShell Module10Z1 Unplugged10Z1 CPAP Device11Tube Adapter11CPAP Tube11End Cap.11Filter11B Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Unpacking and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Ramp Mode18User Software19USB Firmware Upgrade.21Inspections21Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning22Fault Conditions.22Fault Conditions.22Fault Conditions.22Fault Conditions.23Guidance and Manufacturer's Declaration24Electromagnetic Emissions & Immunity25	Battery Warnings and Cautions	8
Package Contents - PowerShell Module 10 Z1 Unplugged 10 Z1 CPAP Device 11 Tube Adapter 11 CPAP Tube 11 End Cap. 11 Filter 11 USB Micro Cable 11 Heat Moisture Exchanger (HME). 11 PowerShell (Optional Accessory) 11 Initial Set Up 12 Unpacking the Z1 Box 12 Unpacking and Powering Up 13 Using Your Z1 15 LCD Display 15 Starting and Stopping Treatment. 17 Programming Ramp Mode 18 User Software 19 USB Firmware Upgrade. 19 Charging the Battery (PowerShell Only). 20 Care 21 Inspections 21 Charging the Replacement . 22 Troubleshooting 23 Guidance and Manufacturer's Declaration 22 Electromagnetic Emissions & Immunity 25	Getting to Know Your Z1 System	.10
Z1 Unplugged 10 Z1 CPAP Device 11 Tube Adapter 11 Tube Adapter 11 CPAP Tube 11 End Cap. 11 Filter 11 USB Micro Cable 11 Heat Moisture Exchanger (HME). 11 PowerShell (Optional Accessory) 11 Initial Set Up 12 Unpacking the Z1 Box. 12 Unpacking and Powering Up. 13 Using Your Z1 15 LCD Display. 15 Starting and Stopping Treatment. 17 Programming Treatment Pressure 18 Programming Ramp Mode 18 User Software. 19 USB Firmware Upgrade. 19 Charging the Battery (PowerShell Only). 20 Care 21 Inspections 21 Cleaning 21 CPAP Tube Replacement 22 Troubleshooting 23 Guidance and Manufacturer's Declaration 23 Electromagnetic Emissions & Immunity 25	Package Contents - Z1 CPAP Base Model	.10
Z1 CPAP Device11Tube Adapter11CPAP Tube11End Cap.11Filter11USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Tatement Pressure18Programming Ramp Mode18User Software19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Package Contents - PowerShell Module	.10
Tube Adapter11CPAP Tube11End Cap.11Filter11USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Assembling and Powering Up13Using Your Z115LCD Display.15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode.19USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Z1 Unplugged	.10
CPAP Tube11End Cap.11Filter11Filter11USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode18User Software.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25		
End Cap.11Filter11USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode18User Software.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Tube Adapter	.11
Filter11USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment17Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration24Electromagnetic Emissions & Immunity25	CPAP Tube	.11
USB Micro Cable11Heat Moisture Exchanger (HME)11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment17Programming Treatment Pressure18Programming Ramp Mode19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Troubleshooting23Guidance and Manufacturer's Declaration25	End Cap	.11
Heat Moisture Exchanger (HME).11PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box.12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Treatment Pressure18User Software.19USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Filter	.11
PowerShell (Optional Accessory)11Initial Set Up12Unpacking the Z1 Box12Assembling and Powering Up13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode19USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning21Charging the Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	USB Micro Cable	.11
Initial Set Up12Unpacking the Z1 Box.12Assembling and Powering Up.13Using Your Z115LCD Display.15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode.18User Software.19USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning.21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Heat Moisture Exchanger (HME)	.11
Unpacking the Z1 Box.12Assembling and Powering Up.13Using Your Z115LCD Display15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode.19USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	PowerShell (Optional Accessory)	.11
Assembling and Powering Up.13Using Your Z115LCD Display.15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode.18User Software.19USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning.21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Initial Set Up	.12
Using Your Z115LCD Display15Starting and Stopping Treatment17Programming Treatment Pressure18Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	Unpacking the Z1 Box	.12
LCD Display15Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Assembling and Powering Up	.13
Starting and Stopping Treatment.17Programming Treatment Pressure18Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	Using Your Z1	.15
Programming Treatment Pressure18Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	LCD Display	.15
Programming Ramp Mode18User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	Starting and Stopping Treatment	.17
User Software19USB Firmware Upgrade19Charging the Battery (PowerShell Only)20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	Programming Treatment Pressure	.18
USB Firmware Upgrade.19Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	Programming Ramp Mode	.18
Charging the Battery (PowerShell Only).20Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions.22Troubleshooting23Guidance and Manufacturer's Declaration25	User Software	.19
Care21Inspections21Cleaning21CPAP Tube Replacement22Fault Conditions22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	USB Firmware Upgrade	.19
Inspections.21Cleaning.21CPAP Tube Replacement.22Fault Conditions.22Troubleshooting.23Guidance and Manufacturer's Declaration.25	Charging the Battery (PowerShell Only)	.20
Cleaning21CPAP Tube Replacement22Fault Conditions22Troubleshooting23Guidance and Manufacturer's Declaration25	Care	.21
CPAP Tube Replacement 22 Fault Conditions. 22 Troubleshooting 23 Guidance and Manufacturer's Declaration 25	Inspections	.21
Fault Conditions22Troubleshooting.23Guidance and Manufacturer's Declaration.25	Cleaning	.21
Troubleshooting	CPAP Tube Replacement	.22
Guidance and Manufacturer's Declaration Electromagnetic Emissions & Immunity	Fault Conditions	.22
Electromagnetic Emissions & Immunity	Troubleshooting	.23
		25
	Technical Specifications	

Performance Specifications	31
PowerShell with Extended Life Battery Specifications	31
Symbol Explanation	32
The following symbols may appear on your device, power supply unit, or packaging $\ldots\ldots$	32
Servicing	33
Limited Warranty	33
Replacement Z1 Parts and Supplies	34
Contact Information	34

Welcome to Your Z1 CPAP System

Thank you for choosing the Z1 CPAP System from Breas.

The Z1 is specially designed with these features:

- The *Z1* motor and firmware combination along with the *Qtube* are designed to minimize both dynamic and static noise at the device and within the mask.
- *Z-Breathe*[®] Comfort Feature *Z-Breathe*[®] is designed to minimize the impact of spikes and drops while keeping the pressure constant, thereby delivering more comfortable therapy.
- Z-Breathe[®] the Z1 automatically reduces the air pressure in the mask to relieve pressure when you exhale and to level out the pressure differences in your natural breathing pattern.
- Leak Compensation the *Z1* automatically compensates for minor air leakage helping ensure the proper treatment pressure is maintained.
- Auto Altitude Adjustment the *Z1* adjusts to altitudes up to 2400 meters above sea level.
- Air Travel The *Z1* adjusts to an airplane's cabin pressure and is approved for use on airplanes.
- Upgradable Operating System– the *Z1* firmware and software can be easily updated via the *USB Cable* and your computer.

This User Guide will help you take advantage of the features and capabilities of this portable *CPAP* device. Please read this User Guide carefully before using your *Z1*.

Owner/User Responsibility

The owner/user of the *Z1 CPAP System* is responsible for reading and understanding this User Guide. The user is responsible for any injury or damage that results from:

- Operation of the *Z1* other than in accordance with the operating instructions contained in the User Guide supplied.
- Unauthorized maintenance or modifications to the device or attached accessories.

Physician Responsibility

The physician is responsible for explaining contraindications and instructions for use to the patients/users.

Replacement Z1 Parts and Supplies

Use only genuine *Z1* replacement filters, parts and supplies. To purchase replacement filters, parts and supplies visit *www.breas.com*. Use of unauthorized filters, parts or replacement supplies can damage the device and void the warranty.

Terms Used in this User Guide

This User Guide contains special terms and icons that appear in the text to draw your attention to specific and important information.

WARNING alerts you to possible injury or hazard.

CAUTION explains special measures for the safe and effective use of the device.

NOTE is an informative or helpful note.

Medical Information

Indications for Use

Your *Z1 CPAP System* is a single user, reusable device that provides continuous positive airway pressure (*CPAP*) to support treatment of adults weighing over 30 kg (66 lbs) with obstructive sleep apnea (OSA).

Contraindications

Positive airway pressure therapy may be contraindicated in some individuals with the following pre-existing conditions:

- Severe bullous lung disease
- Pneumothorax
- Pathologically low blood pressure
- Dehydration
- Cerebrospinal fluid leak, recent cranial surgery, or trauma.

Adverse Effects

Users should report unusual chest pain, severe headache, or increased breathlessness to their prescribing physician. An acute upper respiratory tract infection may require temporary discontinuation of treatment.

The following side effects may arise during the course of therapy with CPAP devices:

- Drying of the nose, mouth, or throat
- Eye irritation
- Nosebleed
- Bloating
- Skin rashes
- Ear or sinus discomfort.

Z1 Warnings and Cautions

A WARNINGS:

- Read the entire User Guide before using your *Z1*. Consult your health care provider if there is any part of this User Guide you do not understand.
- Use your *Z1* and its accessories only for their intended use as described in this User Guide and as directed by your physician, clinician or health care provider.
- The *Z1* is not approved for use with oxygen. Do not use supplemental oxygen or any other type of supplementary gas with your *Z1*. Do not connect supplementary oxygen or other gas to any accessory or patient interface which is connected to your *Z1*.
- If you notice any unexplained changes in the performance of your *Z*1 or your *Z*1 is making unusual or harsh sounds, or if the *Z*1 or *Power Supply* are dropped or mishandled, discontinue use and contact Breas or your health care provider.
- If your *Z1*, *Power Supply, PowerShell* or *Battery* is dropped or mishandled, or if water is spilled into an enclosure, or if an enclosure is broken, discontinue use and contact Breas or your health care provider.
- A mask is not provided. Your *Z1* should only be used with masks and connectors recommended by your physician, clinician, respiratory therapist or health care provider.
- Your Z1 is intended to be used with CPAP masks (or connectors) which have vent holes to allow continuous flow of air out of the mask. When your Z1 is turned on and functioning properly, new air from the Z1 flushes the exhaled air out through the mask vent holes. However, when the Z1 is not operating properly (e.g., at low pressure) or the Z1 is off, insufficient fresh air may be provided through the mask and the exhaled air may be rebreathed. Rebreathing exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This applies to most CPAP devices. Your health care provider will recommend a mask and air tubing to suit your needs. The Z1 supports several CPAP mask types including nasal masks, full face masks, and nasal prongs that meet the ISO 17510-2 standard.
- Do not use your *Z1* with a mask unless the *Z1* is turned on. Once the mask is fitted, confirm that air is flowing from your *Z1*.
- Do not block the vent hole or holes in the mask. Follow the mask manufacturer's instructions for proper use of the mask.
- In the event of a power failure or machine malfunction, remove the mask immediately.
- At low pressures some exhaled gases may remain in the mask.
- **Electrocution Hazard** Your *Z1* is an electrical device, and as such, you should not immerse your *Z1*, *Power Supply, PowerShell* or *Battery* in water. Always unplug your *Z1* and its accessories before cleaning (and clean only as instructed in this User Guide) and make sure your *Z1* and its accessories are dry before plugging back in.
- Before use, make sure the power cord and plug are in good condition and that the cord is not damaged.

A WARNINGS:

- Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps or scented oils to clean your *Z1*, *Power Supply, PowerShell* or *Battery*. These solutions may create a shock hazard, cause damage and reduce the life of these items.
- **Explosion Hazard** Do not use or store your *Z1*, *Power Supply*, *PowerShell* or *Battery* in the vicinity of flammable anesthetics or oxygen.
- Do not use your Z1 near toxic or harmful vapors.
- Do not drop your *Z*¹ or any of its accessories or expose it to any crushing forces. Should your *Z*¹ be dropped and fail to operate properly, contact Breas or your health care provider before using the *Z*¹ for treatment.
- Do not insert any object into any tubing or opening of your *Z1* except as described in this User Guide.
- Do not operate your *Z1* in any area which may block its air intake. Blocking the *CPAP Tube* and/or air inlet of your *Z1* while in operation could lead to overheating of the *Z1*.
- If you put the *Z1* on the floor, make sure the area is free from dust and clear of bedding, clothes or other objects that could block the air inlet or cover the *Power Supply* unit.
- Only store or transport the *Z1* and accessories within the storage and transport temperature, humidity and altitude conditions specified in the Technical Specifications section of this User Guide.
- Do not operate your *Z1*, *Power Supply*, *PowerShell* or *Battery* in an area which may cause overheating.
- Do not cover the *Power Supply*, *PowerShell* or *Battery* with any object or material which would cause them to overheat.
- Keep the power cord, *Power Supply, Battery* and *Z1* away from hot or flammable surfaces, gases or materials.
- Do not open your *Z1* or tamper with its accessories. There are no user serviceable parts inside. Repairs and servicing should only be performed by an authorized Breas service agent.
- Only power the Z1 with the Z1 Power Supply (AC-to-DC power adapter) or the optional Z1 PowerShell with Breas Medical Battery. Do not use any other AC adapters, batteries, or other power sources to power the Z1. Do not use the Z1 Power Supply, PowerShell or Battery to power any device other than the Z1.
- Your *Z1* should only be used with standard CPAP air tubing as recommended by your physician, clinician or health care provider.
- Do not leave long lengths of air tubing around the top of your bed. The tube could twist around your head or neck while you are sleeping.
- Do not connect or insert any device, cable or articles into the Micro USB or Micro SD ports of your *Z1* except as described in this User Guide. Connecting other devices could result in injury to you or damage to the *Z1* and void the warranty.

A WARNINGS:

- Do not locate your *Z1* in an unapproved place or where the air intake can be blocked during treatment. This will lead to overheating of the *Z1*. If a blocked intake should occur during operation, the unit may stop supplying pressurized air and cause a fault condition. Refer to "Troubleshooting" in this User Guide to interpret this fault and instructions for corrective action.
- The *Z1* is designed to deliver pressures from 4 to 20 cmH₂O. In the unlikely event of certain fault conditions, pressures of up to 30 cmH₂O are possible.
- Blocking the *CPAP Tube* while in operation could lead to overheating of the *Z1*.
- In the clinical environment, any personal computer that is used with your *CPAP* system must be at least 1.5 meters (5 ft) away from, or at least 2.5 meters (8 ft) above, the patient. It must also comply with the relevant test standard. For personal computers the international standard is IEC 60950 or equivalent.

♦ CAUTIONS:

- Be careful not to place your *Z1* where it can be bumped or where someone is likely to trip over the power cord.
- Make sure your *Z1*, *Power Supply*, *PowerShell* and *Battery* are kept clean, dry, and free of dust.
- Always make sure the power cord and plugs are in good condition and the equipment is not damaged before use.
- Make sure the area around your Z1 and accessories is dry and clean.
- During partial (below rated minimum voltage) or total power failure, therapy pressures will not be delivered. When power is restored, your *Z1* will enter standby mode and all prior settings will be retained in memory.
- Follow the storage and cleaning instructions provided with your *CPAP Tube* and mask.
- The air filter should be changed at intervals specified in this User Guide. Accumulation of filtered particles over time can reduce the efficiency and shorten the life of any *CPAP* device, including the *Z1*. Replacement filters are available at *www.breas.com*.
- Do not wash the air filter. The air filter is not washable or reusable.
- The *End Cap* protects the *Z1* in the event of a minor collision. Always ensure that the *End Cap* is securely fitted in accordance with instructions at all times. The *Z1* may not operate if the *End Cap* is not properly installed.
- The temperature of the airflow for breathing produced by *CPAP* devices can be higher than the temperature of the room. This is true of all *CPAP* devices. Caution should be exercised when using your *Z1* at room temperatures warmer than 32°C (90°F).
- Only operate the Z1 and accessories within the operating temperature, humidity and altitude conditions specified in the Technical Specifications section of this User Guide.

♦ CAUTIONS:

- Only store or transport the *Z1* and accessories within the storage and transport temperature, humidity and altitude conditions specified in the Technical Specifications section of this User Guide.
- Damage to the *CPAP Tube* may result in air leaks. If this occurs, stop using the damaged *CPAP Tube* and replace immediately.

Battery Warnings and Cautions

The *PowerShell* comes with a removable *Lithium Ion Battery*. Careful attention must be paid to the warnings and cautions described below relating to the *Battery*. Misusing the *Battery* may cause the *Battery* to get hot, rupture, ignite and cause serious injury.

WARNINGS:

- Do not place the *Battery* in fire or heat the *Battery*.
- Do not install the *Battery* backwards so that the polarity is reversed.
- Do not connect the positive terminal and the negative terminal of the *Battery* to each other with any metal object (such as wire).
- Do not carry or store the *Battery* together with necklaces, hairpins, or other metal objects.
- Do not pierce the *Battery* with nails, strike the *Battery* with a hammer, step on the *Battery* or otherwise subject it to strong impacts or shocks.
- Do not solder onto the *Battery*.
- Do not expose the *Battery* to water or salt water, or allow the *Battery* to get wet.
- Do not disassemble or modify the *Battery*. The *Battery* contains safety and protection devices which, if damaged, may cause the *Battery* to generate heat, rupture or ignite.
- Do not place the *Battery* on or near fires, stoves or other high-temperature locations. Do not place the *Battery* in direct sunlight, or use or store the *Battery* inside cars in hot weather. Doing so may cause the *Battery* to generate heat, rupture or ignite. Using the *Battery* in this manner may also result in a loss of performance and a shortened life expectancy.
- Immediately discontinue use of the *Battery* if, while using, charging or storing the *Battery*, the *Battery* emits an unusual smell, feels hot, changes color, changes shape or appears abnormal in any other way. Contact Breas or your health care provider if any of these problems are observed.
- Do not place the *Battery* in microwave ovens, high-pressure containers, or on induction cookware.
- In the event that the *Battery* leaks and the fluid gets into one's eye, do not rub the eye. Rinse well with water and immediately seek medical care. If left untreated the battery fluid could cause damage to the eye.

A WARNINGS:

- Only charge the *Battery* with the supplied *Z1 PowerShell* and the supplied *Z1 Power Supply*. Do not use any other removeable battery charger, AC adapter, or other device to charge the *Battery*.
- Do not attach the *Battery* directly to a vehicle cigarette lighter or to the *Power Supply* plug.
- Do not continue charging the *Battery* if it does not recharge within the specified charging time. Doing so may cause the *Battery* to become hot, rupture or ignite.
- Remove the *Battery* from the *PowerShell* if the *Z1* is not likely to be used for some time. Do not use the *Battery* to power any device other than the *Z1*.
- Only use or charge the *Battery* within the operating temperature, humidity and altitude conditions specified in the Technical Specifications section of this User Guide.
- Only store or transport the *Battery* within the storage and transport temperature and humidity conditions specified in the Technical Specifications section of this User Guide.

♦ CAUTIONS:

• When the *Battery* is worn out, insulate the terminals with adhesive tape or similar materials before disposal.

Getting to Know Your Z1 System

Package Contents - Z1 CPAP Base Model

Description
Z1 CPAP Device
Power Supply with Power Cord
USB A to Micro B Cable
CPAP Tube Adapter
CPAP Tube
End Cap
Filter
Heat/Moisture Exchange Unit (HME)
Z1 CPAP Getting Started Guide
User Guide

Package Contents - PowerShell Module

Description	
Extended Life Battery	
Battery Shuttle	
PowerShell	
PowerShell Getting Started Guide	

Z1 Unplugged

The Unplugged package contains Z1 CPAP base model plus the PowerShell module.

Z1 CPAP Device

The Z1 supplies continuous positive airway pressure and can be programmed to fit the patient's prescriptive needs.

Power Supply (AC to DC Power Adapter)

The *Power Supply* provides power to the *Z*1. It takes AC power (voltage ranging from 100-240 volts) from a wall outlet and converts it to DC voltage to power the *Z*1. The *Power Supply* can be used with international power sources provided the appropriate plug adapter is used.

Tube Adapter

The *Tube Adapter* attaches the *CPAP Tube* to the *Z1*.

CPAP Tube

The *Z1 System* is provided with a *CPAP Tube*. The *CPAP Tube* connects the *Z1* to your mask. Always consult with your physician or health care provider regarding the selection of tubing for your treatment. Follow the instructions provided with the tubing. To purchase replacement tubing visit *www.breas.com*.

End Cap

The *End Cap* attaches to the air intake end of the *Z1* and houses the *Filter*. For proper operation of the *Z1*, the *Filter* should be replaced monthly or more often if necessary.

Filter

The *Z1* uses a specially designed filter. The material has been specifically chosen for the *Z1* and has been designed to be much larger than other *CPAP* filters. The unique shape of the *filter* fits into the *End Cap* of the *Z1*, which was designed to allow for more effective filter surface area.

NOTE: To purchase replacement *Filters* visit *www.breas.com*.

USB Micro Cable

The USB Cable allows the Z1 to communicate with a computer to collect and view user data.

Heat Moisture Exchanger (HME)

The disposable *HME* works by capturing moisture as you exhale and provides it back to you as you breathe. The *HME* does not use any electricity or complex moving parts and avoids the problems encountered with electrical humidification systems.

To install the *HME*, connect the large diameter end of the *HME* to your mask and the smaller diameter end to your *CPAP Tube*. Be sure both connections are fully seated and secure. Store the *HME* in a sealed plastic bag between uses to extend its life. Discard when it is no longer providing humidification.

PowerShell (Optional Accessory)

The *PowerShell* houses both the *Z1* and the *Battery* which allows temporary usage of the *Z1* without the *Power Supply*. To purchase the *PowerShell* visit *www.breas.com*.

Initial Set Up

Unpacking the Z1 Box

You will need to perform a few minor setup steps before using your new **Z1**. The instructions below will guide you through the setup process. Some of the steps outlined below may have been performed by your health care provider prior to delivering the **Z1** to you. If you have any questions or problems as you go through the setup process, please contact Breas or your health care provider for assistance. Breas wants your user experience with the **Z1** to be trouble and frustration free.

- 1. Carefully remove the contents of the box.
- 2. Some of the *Z1* components have their own packaging material. Remove this material carefully. Avoid using anything sharp when opening the packaging.
- 3. Carefully inspect all contents against the Package Contents list on page 10 to assure all components are present. Keep in mind that your health care provider might have completed some of the setup steps prior to delivery, so if a component appears to be missing, check to see if it has already been connected. Also inspect all contents to ensure there is no obvious or visible damage or defects. If you have any concerns or questions, contact Breas or your health care provider before use.
- 4. Place all the components in an area where you can easily access the *Z1* for set up and programming.

WARNING: Do not use this product if you have any concerns about the condition of the contents or if there are any missing parts.

The contents of the Z1 box includes the following:

Z1 Base Model



Z1 Unplugged

Includes everything that comes with the Z1 Base Model, plus:





PowerShell

Battery Shuttle and Battery

Assembling and Powering Up

Filter and End Cap

- 1. Remove the *End Cap* from the *Z1*. Hold the body of the *Z1* in one hand. With the other hand, squeeze firmly on the sides of the *End Cap* and pull it down and away from the *Z1*.
- 2. Discard used Filter.
- 3. Remove the new *Filter* from its packaging.
- 4. Insert the *Filter* into the *End Cap*, aligning the arched side of the *Filter* with the top of the *End Cap*.
- 5. Ensure the Filter covers all holes in the *End Cap*, so that all air entering the *Z1* passes through the *Filter* and no air bypasses the *Filter*.
- 6. Press the *End Cap* onto the *Z1*. (Figure 1)

NOTE: To remove the *End Cap*, squeeze firmly on the sides of the *End Cap* and pull it away from the *Z*1.

Tube Adapter, CPAP Tube, HME and Mask

- Connect one end of the *CPAP Tube* to the round end of the *Tube Adapter* then insert the other end of the *Tube Adapter* into the outlet of the *Z1*. (Figure 2)
- 2. Connect the other end of the *CPAP Tube* to the *HME* and the *HME* to the mask.
- 3. Ensure all connections are tight to avoid air leaks. (Figure 3)

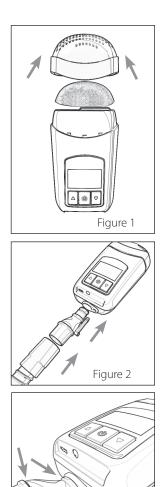


Figure 3

Power Supply

WARNING: The *Z1* must only be used with the *Z1 Power Supply* (or the optional *Z1 PowerShell* with Breas *Battery*). DO NOT use any other power source with the *Z1*.

◆ CAUTION: Make sure the AC power cord IS NOT plugged into an AC power outlet before continuing to the next step.

- 1. Plug the AC power cord into the AC Adapter.
- 2. Plug the AC power cord into a wall outlet.
- 3. Plug the AC Adapter into the Z1. (Figure 4)
- 4. The *Z1* will power up and cycle through a series of startup displays.
- 5. After about 10 seconds, the *Z1* will enter standby mode. The display will read 0.0. (Figure 5)

NOTE: The *Z1* display will be active whenever the device is receiving power.

PowerShell and Battery (Available Separately)

WARNING: The *Z1* must only be used with the optional *Z1 PowerShell* with Breas *Battery* (or the *Z1 Power Supply*). DO NOT use any other power source with the *Z1*.

1. Slide the *Battery* with the attached *Shuttle* into the lower bay of the *PowerShell* until the *Shuttle* clicks into place. (Figure 6)

NOTE: The *Battery* can be removed from the *PowerShell* by pressing the two blue buttons on the *Battery Shuttle* and sliding the *Battery* out.

2. Slide the *Z1* all the way into the upper bay of the *PowerShell* until you hear a "click" sound, indicating the *Z1* has locked into place. (Figure 7)

CAUTION: Insert the Z1 carefully ensuring the power barrel connector slides into the power input on the Z1.

NOTE: The *Z1* can be removed from the *PowerShell* by pressing the two [**b**] symbols on the sides of the *PowerShell* and sliding the *Z1* out.

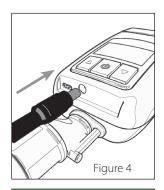








Figure 6



Figure 7


- 3. The *Z1* will power up and cycle through a series of startup displays.
- 4. After about 10 seconds, the *Z1* will enter standby mode. The display will read 0.0. (Figure 8)



NOTE: Remove the *Battery* from the *PowerShell* when the *Z1* is not in use, unless the *Power Supply* is plugged into the *PowerShell* to charge the *Battery*. Otherwise the *Z1* and *PowerShell* will drain the battery completely within 2-3 days.

Using Your Z1

LCD Display

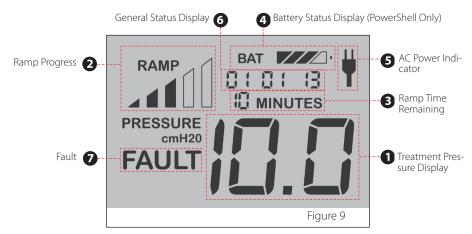

Treatment Pressure Display

The current treatment pressure is shown on the LCD display. Once set, the prescribed treatment pressure will be retained in the Z1's memory even during loss of power. For example, a treatment pressure setting of 10.0 cmH₂O is displayed in Figure 9.

2 Ramp Progress & 3 Ramp Time Remaining Indicators

Ramp Mode allows you to gradually acclimate to the prescribed treatment pressure. Ramp Mode gradually increases the pressure from a low initial starting pressure to the prescribed treatment pressure over a programmed interval of time. See "Programming Ramp Mode".

If you program the *Z1* for Ramp Mode, the Ramp Progress and Ramp Time Remaining indicators will appear on the LCD display. For example, Figure 9 indicates that the Ramp is just over half complete, with a Ramp Time remaining of 10 minutes.

Battery Status Display

This icon is displayed when the optional *PowerShell* and *Battery* are connected. The level of charge remaining on the *Battery* is indicated by the number of solid bars in the icon. The bars flash when the *Battery* is charging. For example, the icon shown in Figure 9 indicates that the *Battery* has between 35-63% of charge remaining. Make sure the *Battery* is fully charged before each use. The bars will stop flashing when the charging is complete.

Number of Solid Bars	Charge Remaining
1	10-35%
2	35-63%
3	63-90%
4	90-100%

6 AC Power Indicator

This icon is displayed when the Z1 is connected to an AC power outlet via the Power Supply.

6 General Status Display

In Standby Mode, this display field alternates between a display of the current date, the current function mode, the *Z-Breathe* setting and the *Auto Stop* feature. During treatment and while setting treatment parameters, this display field shows the currently selected function mode (*CPAP*) and *Z-Breathe* setting (1, 2 or 3).

The General Status Display field may also give notification of certain events which will NOT prevent the *Z1* from operating normally but may need to be addressed by the user.

Code	Event	Resolution	
FAIL 1	Attempt to restart while motor is still running.	Wait at least 10 seconds before attempting to restart treatment again.	
FAIL 2	Attempt to set ramp start pres- sure while ramp time = 0.	 Set ramp time to a minimum of 5 minutes to enable the ability to set ramp start pressure. 	
FAIL 3	Attempt to save treatment pressure while device is locked.	Treatment pressure changes can only be made by a trained healthcare provider while the device is unlocked.	
drift off	Data is no longer getting logged. Therapy will continue.	Internal battery needs replacement, so the device has stopped logging data. Call Breas to arrange for the battery to be replaced. Un- til then therapy will continue to be provided.	


7 FAULT

"FAULT" is displayed if one of several conditions has occurred that require your attention. See "Fault Conditions" (page 22) for explanations of the FAULT codes.

Starting and Stopping Treatment

 Press the Start/Stop Button on the Z1 to initiate treatment using the programmed treatment setting. The Treatment Pressure Display will show the actual pressure being delivered at all times. If the Ramp Mode has been programmed, the Ramp start pressure will be displayed upon pressing the Start/Stop Button and the indicated pressure will rise as the Z1 Ramps up to prescribed treatment pressure. If the incorrect pressure is displayed, contact your health care provider before commencing CPAP therapy. (Figure 10)

CAUTION (When using the PowerShell Only): Make sure your Battery is fully charged before commencing CPAP therapy using the PowerShell. The battery life of a fully charged Extended Life Battery in the PowerShell is typically 8.9 hours or longer and treatment will terminate when the charge is expended.

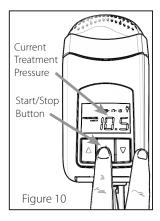

2. Press the *Start/Stop Button* a second time to stop treatment.

Auto Stop Feature

The *Z1* is equipped with an *Auto Stop Feature*. This will stop the treatment automatically after the mask has been removed. To restart the treatment, place the mask back on your face and press the *Start/Stop Button*.

NOTE: The Auto Stop Feature will stop the therapy if a large air leak is detected between the *Z*1 outlet and the mask. This may also happen at times when the mask is not being worn properly. This common condition should be resolved once all connections are secure, the mask is properly worn, and the Start/Stop Button has been pushed.

Setting the Auto Stop Feature

If you prefer to stop the *Z*1 using the *Start / Stop Button*, you can disable the *Auto Stop Feature*. To do this press and hold the *Up Arrow Button* and *Down Arrow Button* at the same time until the *Z-Breathe* setting flashes. Release the buttons. Then press and hold the *Up Arrow Button* and *Down Arrow Button* at the same time again until *RDFF-DT* flashes. Release both buttons and press the *Down Arrow Button* to change to *RDFF-DFF*. Press and hold the *Start / Stop Button* to save the setting. To enable the feature, repeat the steps above, but press the *Up Arrow Button* to change to *RDFF-DT*.

Programming Treatment Pressure

When the *Z1* is in standby mode, press and hold the *Start/Stop Button* for 3 seconds until the pressure value flashes. While it is flashing, repeatedly press either the *Up* or *Down Arrow Buttons* to increase or decrease the pressure (Figure 11). When you have reached the prescribed pressure, press and hold the *Start/ Stop Button* for approximately 3 seconds to save the treatment pressure setting (this setting will be saved even when the *Z1* is not powered). The *Z1* will return to standby mode. To return to standby mode without saving the new setting, press and release the *Start/ Stop Button* momentarily instead of holding it down.




Programming Ramp Mode

Some users find it comforting to acclimate to *CPAP* therapy each session. This can be accomplished by programming the Ramp Mode which allows you to start with a low initial pressure and gradually increase to the prescribed treatment pressure.

Setting Ramp Time

When the *Z1* is in standby mode, press and hold the *Up Arrow Button* for approximately 3 seconds until the "00 MINUTES" value flashes. While it is flashing, repeatedly press either the *Up* or *Down Arrow Buttons* to increase or decrease the Ramp Time in 5 minute increments. (Figure 12) Set the Ramp Time to "00 MINUTES" to disable the Ramp Mode.

When the desired Ramp Time value is reached, press and hold the *Start/Stop Button* for approximately 3 seconds to save your Ramp Time setting (this setting will be saved even when the *Z1* is not powered). The *Z1* will return to standby mode. To

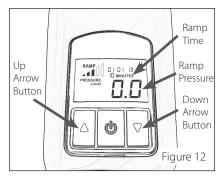

return to standby mode without saving the new setting, press and release the *Start/Stop Button* momentarily instead of holding it down.

NOTE: Ramp Time must be set to a minimum of 5 minutes prior to attempting to program Ramp Start Pressure or the change to Ramp Start Pressure will not take effect and "FAIL 2" will be shown in the General Status Display field.

Setting Ramp Start Pressure

Press and hold the *Down Arrow Button* for 3 seconds until the Ramp start pressure value flashes. While flashing, repeatedly press either the *Up* or *Down Arrow Buttons* to increase or decrease the Ramp start pressure by 0.5 cmH₂O increment. (Figure 11) Ramp start pressure must be lower than prescribed pressure.

When the desired value for Ramp start pressure is reached, hold down the *Start/Stop Button* for approximately 3 seconds to save your setting. The *Z1* will return to standby mode. To return to standby mode without saving the new setting, press the *Start/Stop Button* momentarily instead of holding it down.

User Software

See separate *Z1* Software Guide for information on data viewing software. This guide may only be available online.

USB Firmware Upgrade

A *Z*¹ firmware upgrade can be downloaded via the Patient Software. To upgrade the firmware on your *Z*¹ connect the *Z*¹ to your PC with a USB cable. The USB firmware upgrade can also be used to set the date and time on the *Z*¹.

- 1. To upgrade firmware, click the "Upgrade Firmware" button.
- 2. To set the date and time click the "Set" button.

Z-Breathe® Algorithm Selection


Z-*Breathe* is a proprietary algorithm engineered in the *Z*¹ to smooth out the breathing cycle while enjoying the benefits of *CPAP* therapy. Typically users experience pressure swings while breathing on a fixed pressure *CPAP* device. Pressure swings occur as a spike in pressure during exhalation and a drop in pressure during inhalation. *Z*-*Breathe* is designed to eliminate the spikes and drops while keeping the pressure constant, thereby delivering more comfortable therapy.

Pressure swings can vary between users and this is due to the lung capacity of the person breathing on the machine. Larger individuals will typically have greater lung capacity that will require a more aggressive setting and for this reason we have created 3 levels of *Z*-*Breathe*:

- CPAP 1 offers mild pressure swing relief
- CPAP 2 is the *Default Setting* and offers moderate pressure swing relief
- CPAP 3 is the most aggressive setting for pressure swing relief

You can change the setting of the *Z*-Breathe Algorithm when your *Z*1 unit is in Standby Mode (pressure of "0.0" indicated on the display). To change the setting, hold down both the *Up* and *Down Arrow Buttons* at the same time until you see "*CPAP* – 1", "*CPAP* – 2" or "*CPAP* – 3" flashing on the display. Press the *Up* or *Down Arrow Buttons* until you observe your desired setting on the display. To lock-in the setting, hold down the *Start/Stop Button* until the "*CPAP* – X" indicator no longer flashes on the display.

The Z-Breathe settings in relation to the fixed pressure are illustrated in the following graph. We encourage you to experiment with the settings to achieve maximum comfort.

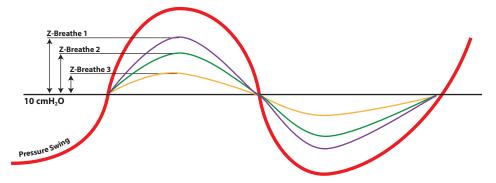

Charging the Battery (PowerShell Only)

WARNING: Only charge the *Battery* with the supplied *Z1 PowerShell* and the supplied *Z1 PowerShell* and the supplied *Z1 Power Supply*. Do not use any other battery charger, AC adapter, or other device to charge the *Battery*.

With both the *Z1* and the *Battery* fully inserted into the *PowerShell*, plug the *Power Supply* into the *PowerShell* as shown and into a power outlet. (Figure 13)

ly inserted into the into the *PowerShell* as irre 13) ur *Battery* is fully g *CPAP* therapy ittery life of a fully *rShell* is typically

◆ CAUTION: Make sure your Battery is fully charged before commencing CPAP therapy using the PowerShell. The battery life of a fully charged battery in the PowerShell is typically 8.9 hours or longer, after which treatment will terminate. To recharge the Battery, see "Charging the Battery"

Ensure the *Z1* is in standby mode (display reads 0.0). The *Battery* will not charge unless the *Z1* is in standby mode. The *Battery* status indicator on the *Z1* will flash when the *Battery* is charging. The number of solid bars indicates the current level of charge. (Figure 14)

The battery status indicator will stop flashing when the battery is fully charged. Typically it will take 12 hours to fully charge the battery. After a few minutes, the back lit display will turn off. This is to conserve power. It does *not* indicate that the battery is fully charged.



Figure 13

NOTE: The battery will not charge if you are using the device at the same time.

Care

The *Z1 System* needs periodic care. It is the user's responsibility to be aware of the following items for care:

- Inspections
- Cleaning
- Filter Replacement
- CPAP Tube Replacement
- Fault Conditions

Inspections

Before each use carefully inspect the *Z1*, *CPAP Tube*, *Filter*, and mask for signs of wear, damage or discoloration. For replacement parts visit *www.breas.com*.

Cleaning

Regular cleaning should be performed in accordance with the instructions below.

WARNING: This device can expose you to electrocution. Do not immerse the *Z1*, *Power Adapter, Power Cord* or *Battery* in water. Always unplug these devices before cleaning (only as instructed in this User Guide) and make sure they are dry before plugging back in.

CAUTION: Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps or scented oils to clean the *Z1*. Never attempt to clean the *Power Adapter, PowerShell, Battery* or accessories with any of these solutions. These solutions may create a shock hazard, cause damage and reduce the life of these products.

Daily

- 1. Remove the *CPAP Tube* and the mask from the *Z1*.
- 2. Refer to the instructions for cleaning, maintenance and replacement provided with your *CPAP Tube* and mask.
- 3. Before next use, reconnect the *CPAP Tube* and the *Tube Adapter* to the *Z1* and mask.

As Needed

1. Clean the exterior of the *Z*¹ with a soft damp cloth and mild detergent and wipe clean. Never submerge the *Z*¹ in water. Do not allow soap or water to enter the *Z*¹.

Filter Replacement

The *Filter* should be inspected weekly for holes and blockage by dirt and dust, and should be replaced every month (or more often if necessary). The *Filter* is accessed by removing the *End Cap* as described in "Assembling and Powering Up: *Filter* and *End Cap*".

NOTE: The *Z1* uses a specially designed Filter that is only available from Breas. To purchase replacement filters visit *www.breas.com*.

WARNING: Do not attempt to use any air filter which is not supplied by Breas for the *Z*1. The *Z*1 is designed to only work with approved materials. A filter which is not approved may cause damage to the *Z*1 from overheating.

CAUTION: Do not operate the *Z1* without the *Filter*. The *Z1* must operate with a filter installed; failure to do so may damage the *Z1* and allow impurities into the respiratory system.

CPAP Tube Replacement


The CPAP Tube should be replaced according to the instructions provided with the CPAP Tube.

Fault Conditions

The *Z1* has the ability to detect operational problems and will display the word FAULT on the LCD screen. (Figure 15)

The LCD screen should be checked for any FAULT messages and visible damage daily.

A fault condition is indicated when the word "FAULT" is displayed accompanied by a two digit flashing number in place of the pressure reading.

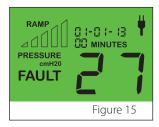

The table below provides a listing of fault codes and recommended actions to be taken by the user.

Fault Code	Explanation	Action
21 Flashing	Treatment pressure not programmed	Discontinue use and contact your health care provider or Breas.
27 Flashing	Overheating	Discontinue use and contact your health care provider or Breas.
32 Flashing	Over current	Discontinue use and contact your health care provider or Breas.
33 Flashing	Over pressure	Discontinue use and contact your health care provider or Breas.

Troubleshooting

Problem	Possible Causes & Solutions		
No Display	Power is not present. Ensure the <i>Power Supply</i> is connected and the power outlet has power.		
	Loose connection of the <i>Power Supply</i> at the <i>Z1</i> or loose connec- tion of the <i>Power Cord</i> at the AC Adapter. Check the connections.		
	<i>Z1</i> is not locked into position in the <i>PowerShell</i> . Slide the <i>Z1</i> into the <i>PowerShell</i> until it clicks and locks into place.		
	<i>Battery</i> is not locked into position in the <i>PowerShell</i> . Slide the <i>Battery</i> into the <i>PowerShell</i> until the <i>Battery Shuttle</i> clicks and locks into place.		
	Battery is not charged. Charge the Battery.		
	A power supply not designed for the <i>Z1</i> is connected to the <i>Z1</i> . Remove the power supply unit and replace with a Breas approved unit.		
Treatment Will Not Start	Check the display for a fault condition. Refer to the Fault Codes table in "Fault Conditions". Contact Breas or your health care provider.		
Treatment Shuts Off	Treatment automatically turns off when the mask is removed. Replace the mask and push the <i>Stop/Start</i> button. If problem con- tinues check for leaks in the connections between all of the pieces between the mask and the unit. Or turn off the <i>Auto Stop Feature</i> .		
Insufficient Air Flow	Air filter is dirty or clogged. Replace air filter.		
Delivered from Z1	Air intake is blocked. Clear blockage.		
	<i>CPAP Tube</i> is not connected properly. Check <i>CPAP Tube, Tube Adapter er</i> and connections to your <i>Z1</i> and your mask.		
	Ramp Mode is in use. Wait for air pressure to reach prescribed treat- ment pressure or stop treatment and adjust Ramp time.		
	<i>CPAP Tube</i> is blocked, pinched or punctured. Unblock <i>CPAP Tube</i> . Check the <i>CPAP Tube</i> for punctures. Replace a damaged <i>CPAP Tube</i> immediately.		
	Mask and/or headgear are not positioned correctly and mask is leak- ing. Adjust position of the mask and headgear until leak ceases.		
	Incorrect <i>CPAP Tube</i> selected. Do not use tubing which was not recommended by your clinician.		
	Check the display for a fault condition. Refer to fault codes in "Fault Conditions".		

Problem	Possible Causes & Solutions		
Pressure Rises Inappropriately	Talking, coughing or breathing in an unusual manner. For exam- ple, avoid talking with a nasal mask on and breathe as normally as possible.		
	There may be an air leak caused by improperly connected tubing. Check for leaks.		
	Follow mask manufacturer's instructions.		
Mask Cushion is Vibrating Against the Skin and an	Stop treatment. Remove and reposition the mask and, if necessary, adjust the headgear. Resume treatment.		
Air Leak is Occurring	Follow mask manufacturer's instructions.		
Noise from Mask	Please note that the <i>Z1</i> (like other small <i>CPAP</i> devices) has a very small motor that turns much faster than the <i>CPAP</i> you have at home to achieve the same therapeutic pressure. With some mask types it is possible that a different sound may be generated from the mask due the proximity of the vent holes and the velocity of air coming up the tube. Most users who have experienced this consider it to be white noise and a trade off for the convenience of the worlds smallest and lightest <i>CPAP</i> . If you experience this and it is a problem for you there are three steps that you can try to help minimize this:		
	Place the <i>HME</i> between the end of your tube and the mask.		
	Try a standard bore (22mm) <i>CPAP Tube</i> . Often these larger bore tubes come with your home device and they are approved to use with your <i>Z1</i> .		
	Use a stand alone Heated Humidifier with your <i>Z1</i> while at home. The volume in the water chamber will greatly reduce any mask noise.		
<i>Z1</i> Has Been Left in a Hot Environment	Allow $Z1$ to cool before use. Disconnect the power cord and then reconnect it to restart the $Z1$.		
Electromagnetic Compability	The <i>Z1</i> and <i>PowerShell</i> meet all applicable electromagnetic compatibility standards. However, if you suspect performance of the device is affected by interference from wireless sources, move the <i>Z1</i> and <i>PowerShell</i> further away from those sources.		

Guidance and Manufacturer's Declaration Electromagnetic Emissions & Immunity

Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to EMC information provided in this document.

Z1 system does not have Essential Performance within the meaning of IEC 60601-1. However, immunity testing was conducted in accordance with IEC 60601-1-2 and compliance was confirmed as summarized in this document.

Guidance a	Guidance and manufacturer's declaration - electromagnetic emissions			
The <i>Z1 CPAP</i> System is intended for use in the electromagnetic environment specified below. T customer or the user of the <i>Z1 CPAP</i> System should assure that it is used in such an environment				
Emissions test	Compliance	Electromagnetic environment — guidance		
RF emissions CISPR 11	Group 1	The <i>Z1 CPAP</i> System uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interfer-		
		ence in nearby electronic equipment.		
RF emissions	Class B	The <i>Z1 CPAP</i> System is suitable for use in all establishments, including domestic establish-		
CISPR 11		ments and those directly connected to the public		
Harmonic emissions	Class A	low-voltage power supply network that supplies buildings used for domestic purposes.		
IEC 61000-3-2				
Voltage fluctuations/ flicker emissions	Complies			
IEC 61000-3-3				

A WARNINGS:

- The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.
- The use of cables and accessories (eg, humidifiers) other than those specified for the device is not recommended. They may result in increased emissions or decreased immunity of the device.

Guidance and manufacturer's declaration — electromagnetic immunity

The *Z1 CPAP* System is intended for use in the electromagnetic environment specified below. The customer or the user of the *Z1 CPAP* System should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic envi- ronment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/out- put lines	±2 kV for power supply lines ±1 kV for input/out- put lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U _T (>95 % dip in U _T) for 0,5 cycle 40% U _T (60% dip in U _T) for 5 cycles 70% U _T (30% dip in U _T) for 25 cycles <5% U _T (>95 % dip in U _T) for 5 sec	<5 % U _T (>95 % dip in U _T) for 0,5 cycle 40% U _T (60% dip in U _T) for 5 cycles 70% U _T (30% dip in U _T) for 25 cycles <5% U _T (>95 % dip in U _T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment, If the user of the <i>Z1 CPAP</i> System requires continued oper- ation during power mains interruptions, it is recom- mended that the <i>Z1 CPAP</i> System be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency mag- netic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: U_{T} is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration — electromagnetic immunity				
The <i>Z1 CPAP</i> System is intended for use in the electromagnetic environment specified below. The customer or the user of the <i>Z1 CPAP</i> System should assure that it is used in such an environment.				
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment, guidance	
			Portable and mobile RF communi- cations equipment should be used no closer to any part of the <i>Z1 CPAP</i> System, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.	
			Recommended separation distance	
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz	<i>d</i> = 1,2√P	
Radiated RF IFC 61000-4-3	3 V/m	3 V/m	$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz	
IEC 61000-4-3	80 MHz to 2,5 GHz	80 MHz to 2,5 GHz	d = 2,3√P 80 MHz to 2,5 GHz	
			where <i>P</i> is the maximum output power rating of the transmitter in watts (W) ac- cording to the transmitter manufacturer and <i>d</i> is the recommended separation distance in metres (m). Field strengths from fixed RF transmit- ters, as deter-mined by an electromag- netic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol:	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				
phones and land cannot be predict to fixed RF transm field strength in the compliance level abnormal perform or relocating the a	mobile radios, amateu ted theoretically with hitters, an electromagr he location in which t above, the <i>Z1 CPAP</i> Sy nance is observed, ado <i>Z1 CPAP</i> System.	ur radio, AM and accuracy. To asse hetic site survey he <i>Z1 CPAP</i> Syste ystem should be ditional measure	ations for radio (cellular/cordless) tele- FM radio broadcast and TV broadcast ess the electromagnetic environment due should be considered. If the measured em is used exceeds the applicable RF observed to verify normal operation. If as may be necessary, such as re-orienting engths should be less than 3 V/m.	

Recommended separation distances between portable and mobile RF communications equipment and the Z1 CPAP System

The *Z1 CPAP* System is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the *Z1 CPAP* System can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the *Z1 CPAP* System as recommended below, according to the maximum output power of the communications equipment.

Rated maximum	Separation distance according to frequency of transmitter m		
output power of transmitter W	150 kHz to 80 MHz <i>d</i> = 1,2√P	80 MHz to 800 MHz <i>d</i> = 1,2√P	800 MHz to 2,5 GHz <i>d</i> = 2,3√P
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

WARNING: Please note that mobile RF communications equipment can effect medical electrical equipment.

Wireless Quality of Service for Z1 Bluetooth Communication with Z1 Mobile App

The *Z***1** and *Mobile App* utilize only a very small fraction of the available bandwidth and throughput. Based on this, the *Z***1** and *Mobile App* are highly tolerant of reductions in wireless quality of service and performance will not be affected until service is essentially lost; Wireless connectivity is not part of the essential performance of the *Z***1** performance. All functions of the *Z***1** can be performed on the *Z***1** unit itself; The *Mobile App* and *Z***1** are tested for coexistence with other inband radio frequency sources; The Bluetooth hardware module is FCC listed, and complies with FCC PART 15 SUBPART C.

IC Notice

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.

2. This device must accept any interference, including interference that may cause undesired operation of the device.

FCC Notice

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

European Conformity according to the Radio Equipment Directive (RED)

Hereby, Breas Medical AB declares that the radio equipment type included in this device is in compliance with Directive 2014/53/EU.

The full text of the EU declaration of conformity is available at the following internet address: www.breas.com/declarations.

INDUSTRY CANADA NOTICE

This device complies with Industry Canada RSS-210 (Rev.8). Operation is subject to the following two conditions

1) this device may not cause interference, and

2) this device must accept any interference, including interference that may cause undesired operation of the device.

PAN131x is licensed to meet the regulatory requirements of Industry Canada (IC), license: IC 216Q-1316.

Technical Specifications

30W Power Adapter

Operating temperature Operating humidity Operating altitude Operating atmospheric pressure Storage and transport temperature Storage and transport humidity Storage and transport altitude Storage and transport atm. pressure Electromagnetic compatibility Aircraft use

IEC 60601-1 classification Available modes

Deliverable pressure accuracies Sound pressure level Nominal dimensions (L x W x H) Weight Housing construction Supplemental oxygen Standard air filter Air outlet PowerShell Battery Specifications Unit Input range 100–240V, 50–60Hz Typical power consumption 20W (30VA) Maximum power consumption 35W (70VA) 5°C to 35°C (41°F to 95°F) 10-80% Relative Humidity, Non-Condensing Sea level to 2400 m (8000 ft) 101 to 75.3 kPa -20°C to +60°C (-4°F to +140°F) 10-95% Relative Humidity, Non-Condensing No limit No limit Compatible with IEC 60601-1-2 Complies with RTCA DO-160G Standard for use in flight Class II (double insulation), Type BF Standby, CPAP, Auto-CPAP and CPAP/Auto-CPAP with Ramp ± 0.6 cm H₂O $\pm 4\%$ of the measured reading < 26 dBA 16.46 cm x 8.38 cm x 5.13 cm (6.48" x 3.30" x 2.02") 284 g (10 oz.) Flame retardant engineered thermoplastic Not for use with oxygen Polyester The 22 mm air outlet complies with ISO 5356-1 99.4 Watt-hours capacity Lithium Ion Battery Pack, 4S2P Weight approx. 440 g (15.5 oz.)

Performance Specifications

Operating pressure range4 to 20 cm H_2O Maximum single fault steady pressure state30 cm H_2O Deliverable pressure accuracy ± 0.6 cm $H_2O \pm 4\%$ of the measured reading

PowerShell with Extended Life Battery Specifications

Longevity (typical operating time)*

Z-Breathe Setting	Treatment Pressure	Operating Time
3	$20 \text{ cm H}_2\text{O}$	8.9 h

* Results reflected were obtained via bench testing under simulated operating conditions at the highest machine settings. Actual run time may vary due to machine settings, machine pressure and personal breathing. Lower pressures will result in longer battery operating times.

Capacity	99.4 watt-hours
Life time	approximately 300 charge/dischrage cycles
Shelf life	Batteries are shipped with between 30% and 50% rated capacity, providing a minimum of 6 months shelf life when stored at 25°C. If storage temperature exceeds 25°C over 6-month period then shelf life will be reduced and battery should be recharged periodically.
Storage and transport temperature	-20°C to +60°C (-4°F to +140°F)
Storage and transport humidity	10-95% Relative Humidity, Non-Condensing
Battery Technology	Lithium Ion Battery Pack, 4S2P
Weight	Approx. 440 g (15.5 oz.).
Charging	Battery charges when Z1 not in use

Symbol Explanation

The following symbols may appear on your device, power supply unit, or packaging.

Φ	Start/Stop
SN	Serial number
REF	Catalog number
LOT	Batch code
\triangle	Caution
	Class II equipment
*	Type BF equipment
X	Product disposal warning according to EN50419
····	Manufacturer
\sim	Date of Manufacture: YYYY-MM
	Read user instructions
IP22	Degree of protection provided by enclosure.
	Direct current
\bigcirc	Not for use on more than one patient
CE 2797	CE marking applies in accordance with the directive MDD 93/42/EEC
c Us	Certified by Curtis-Straus
FCC ID	FCC Identifier

Servicing

The *Z1 System* is intended to provide safe and reliable operation when operated in accordance with the instructions provided by Breas. Breas recommends that the *Z1 System* be inspected and serviced by an authorized Breas Service Center if there is any sign of wear or concern with *Z1* function. Otherwise, service of the *Z1* generally should not be required during the warranty period (see 'Limited Warranty' for details) of the device.

Limited Warranty

Breas warrants that your Breas product shall be free from defects in material and workmanship from the date of purchase for the period specified below.

Component	Warranty Period	
Z1 device	3 years	
PowerShell	1 year	
Extended Life Battery	6 months	
Power Supply	1 year	

If product fails to perform in accordance with product specifications, Breas will repair or replace, at its option, the defective material or part. This warranty is only available to the initial purchaser. It is not transferable. If the product fails under conditions of normal use, Breas will repair or replace, at its option, the defective product or any of its components. This limited warranty does not cover: a) any damage caused as a result of accident, improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organization that has not been expressly authorized by Breas to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) damage caused by water entering the product. Warranty claims on defective product must be made by the initial purchaser by contacting Breas.

THIS WARRANTY REPLACES ALL OTHER EXPRESSED OR IMPLIED WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. SOME REGIONS OR STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU.

BREAS MEDICAL SHALL NOT BE RESPONSIBLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES CLAIMED TO HAVE RESULTED FROM THE SALE, INSTALLATION OR USE OF ANY BREAS MEDICAL PRODUCT. SOME REGIONS, STATES OR COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact Breas or your health care provider.

Replacement Z1 Parts and Supplies

Use only genuine Z1 replacement filters, parts and supplies. Replacement filters, parts and supplies can be purchased by visiting www.breas.com.

Product	Description
Z1® Base CPAP System	Z1 Base System includes: Z1 Device, Tube Adapter, Slim-Style Tube, Power Supply, Custom USB Cable, User Manual and Getting Started Guide.
Z1 Unplugged® CPAP System	Z1 Unplugged System includes: Z1 Base System and PowerShell Module.
Z1 PowerShell® Module	Z1 PowerShell includes: PowerShell Battery Cradle, Neoprene Cover & Lanyard, Extended Life Battery and Battery Shuttle.
Z1 Battery Module	14.4V rechargeable lithium ion Extended Life Battery pack for the Z1 PowerShell.
Z1 Custom USB Cable	USB to micro USB cable with custom tip for software downloads and data extraction.
Slim-Style Tube	Special lightweight, thin pressure delivery tube for use with the Z1 Device.
Z1 Polyester Filter (2 pack)	2 each, Z1 Polyester Air Filters
Z1 Polyester Filter (6 pack)	6 each, Z1 Polyester Air Filters
Z1 Custom Tube Adapter	Z1 custom outlet Tube Adapter (required for Z1 operation)
Heat/Moisture Exchange Unit (HME)	In-line humidification for the Z1 Device
Z1 Power Supply	AC Adapter, 100 - 240V
Z1 Power Cord, EU	AC Power Cord with EU Plug
Z1 Power Cord, UK	AC Power Cord with British Plug
Z1 Power Cord, AU	AC Power Cord with Australian Plug
Z1 Plastic Filter End Cap	Z1 Filter Housing and Cover

Contact Information

Breas Medical AB Företagsvägen 1 SE 435 33 Mölnlycke SWEDEN www.breas.com Phone: Order: Technical Support: Email:

BREAS

+46 31 86 88 00 +46 31 86 88 20 +46 31 86 88 60 breas@breas.com

BREAS

FAA Compliance

To whom it may concern:

The US Department of Transportation (DOT) Final Rule, "Nondiscrimination on the Basis of Disability in Air Travel" (73 FR 27614 which updates Title 14 CFR Part 382), effective May 13, 2009 provides important requirements for the accomodation of passengers with respiratory assistive devices (Ventilators, Respirators and CPAP machines).

In line with these requirements, respiratory assistive devices may be used onboard an aircraft, without further testing by the carrier, provided they have been tested for Electromagnetic Compatibility (EMC) in accordance with the current version of RTCA/DO-160, Section 21, Category M.

Breas Medical has successfully completed testing for the Z1 CPAP System. The Z1 CPAP System complies with RTCA/DO-160, Section 21, Category M and can be considered FAA compliant.

Some airlines may require advance notification before travel, and devices may need to be operated by battery. Breas Medical recommends that customers check with their airline.

