


LCD Display Overview

The **Z1** display offers useful information on device settings. Below is a brief overview of the information available using the built-in LCD. For more information, please refer to the **Z1** User Guide.

1. Treatment pressure display
2. Ramp progress indicator (if ramp set)
3. Ramp time remaining (if ramp set)
4. Battery status (if present)
5. AC Power indicator (if connected)
6. General Status Display
7. Fault indicator (if fault occurs)

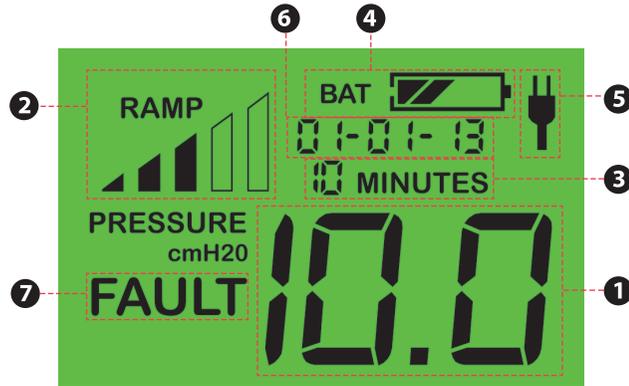

NOTE: Code **ERR OFF**: Data is no longer getting logged. Therapy will continue. Internal battery needs replacement, so the device has stopped logging data. Call Breas to arrange for the battery to be replaced. Until then therapy will continue to be provided.

If you encounter problems consult the **Z1** User Guide or contact your health care provider or Breas.



Z1™ is a trademark of Human Design Medical, LLC.
Patent Pending. © Copyright 2013, Breas Medical AB.
All rights reserved.

BREAS



Z1 Getting Started Guide
Doc. HD40-0005-02 En Y-1

Z1™ and Z1™ Auto Getting Started Guide



Welcome!



Congratulations on choosing the world's smallest, lightest, most portable CPAP machine! Your new Z1 is almost ready for use. To begin using your Z1, follow the steps outlined below. Please read the entire Z1 User Guide.


1. Unpack the filter. Insert the filter into the end cap as shown, with arched side of the filter aligned with the top of the end cap. Ensure the filter covers all holes in the end cap. Press the end cap onto the Z1.



2. Connect one end of the CPAP tube to the round end of the tube adapter. Insert the other end of the tube adapter into the outlet of the Z1 as shown. Connect the other end of the CPAP tube to the mask. Ensure all connections are tight to avoid air leaks.



3. Plug the power supply (AC to DC power adapter) into the Z1 as shown and into a wall outlet. The Z1 will power up in standby mode. The display will read 0.0. The Z1 display will always be active whenever the device is receiving power.

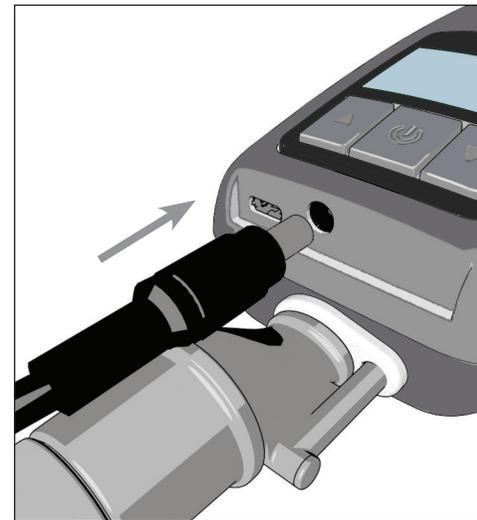

4. Program the prescribed treatment pressure following the instructions in the User Guide. Push the **Start/Stop Button** (⏻) to initiate therapy. Push this button a second time to stop therapy.



NOTE for Z1: The *Auto Stop Feature* will stop the therapy if a large air leak is detected between the Z1 outlet and the mask. This may also happen at times when the mask is not being worn properly. This common condition should be resolved once all connections are secure, the mask is properly worn, and the *Start/Stop Button* has been pushed.

NOTE for Z1 Auto: The *Auto Start/Stop Feature* will stop the therapy if a large air leak is detected between the Z1 outlet and the mask. This may also happen at times when the mask is not being worn properly. This common condition should be resolved once all connections are secure, the mask is properly worn, and the *Start/Stop Button* has been pushed or you start to breathe through the mask.

NOTE: Ramp time and initial ramp pressure can be adjusted while the Z1 is in standby mode (screen displays 0.0). To set the ramp time, press and hold the **Up Arrow** (▲) until the ramp time flashes. To set the initial ramp pressure, press and hold the **Down Arrow** (▼) until the pressure display flashes. Repeatedly press the **Up** or **Down Arrow** until the desired setting is reached. (Set ramp time to "00" to disable the ramp feature.) Press and hold the **Start/Stop Button** (⏻) to save the new settings. For more information on adjusting settings see the Z1 User Guide.